How to use your new Rehband Support - 7752

RU

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UK

Function

Gives support and warmth. Great flexibility to the knee joint. Protects the knee from blows.

Indications

Knee support when playing handball.

Material

5 mm SBR/Neoprene, polyamidejersey, Darlexx®, Kevlar®, LD-Polyethan with EVA

Application

- 1) Place the support on the knee.
- 2) The kneecap must lie in the intended groove inside the pelotte.

Important Information

- 1) If you have questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/248°F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber* or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) Re: SBR/Neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- 6) If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- 7) If you are allergic to heat, we advise you to use a product that not contains SBR/Neoprene.

*Re: Medical- products, the prescriber must be a qualified doctor, PT, OT or Orthopaedic Engineer.

Washing Instructions

- 1) Close all Velcro.
- 2) Remove detachable splints.
- 3) Mesh laundry bag is recommended.
- 4) Stretch product while wet and let dry

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